

# Artron One Step Cassette Format Urine Pregnancy Test

Catalog No. A01-C

## INTENDED USE

The *Artron One Step Cassette Style HCG Urine Pregnancy Test* used for qualitative determination of human chorionic gonadotropin (hCG) in urine for early detection of pregnancy.


## SPECIMEN COLLECTION AND PREPARATION

- Any urine specimen is appropriate for hCG Testing but the first morning urine specimen is optimal because of its highest concentration of hCG.
- Urine specimens may be collected in any clean, dry, plastic or glass container.
- If specimens cannot be assayed immediately, they may be stored at 2-8°C for up to 48 hours prior to testing.
- Specimen should be allowed to reach room temperature before testing.

## TEST PROCEDURE

Test must be in room temperature (18°C to 30°C)

1. Remove a Testing Device from the foil pouch by tearing at the "notch" and place it on a level surface.
2. Holding a Sample Dropper vertically, add exactly four drops of the urine specimen to the sample well (with an arrow marked)
3. Read results at five minutes.

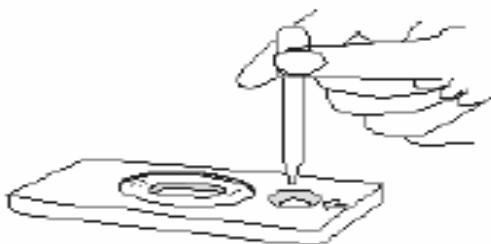


## INTERPRETATION OF RESULTS

**Negative:** Only one color band appears on the control region. No apparent band on the test region. This indicates that no pregnancy has been detected.

**Positive:** Distinct color bands appear on the control and test regions. It indicates that you are pregnant. The color intensity of the test bands may vary since different stages of pregnancy have different concentrations of HCG hormone.

**Invalid:** No visible band at all. Repeat with a new test kit. If test still fails, please contact the distributor with the lot number.

## TEST RESULTS

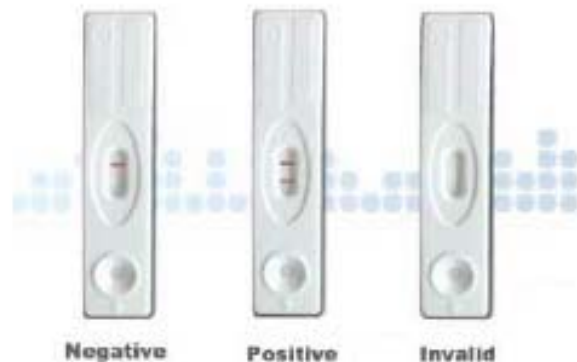

## STORAGE AND STABILITY

The test kits can be stored at room temperature (18°C to 30°C) in the sealed pouch up to the expiration date. The test kits should be kept away from direct sunlight, moisture and heat.

## PRECAUTION

1. For in vitro diagnostic use only.
2. Do not use test kit beyond expiry date.
3. The test device should not be reused.

**Artron BioResearch Inc.**

3938 North Fraser Way, Burnaby BC V5J 5H6 Canada

TEL: 1 604 415 9757 FAX: 1 604 415 9795 Website: [www.artronbio.com](http://www.artronbio.com) Email: [info@artronbio.com](mailto:info@artronbio.com)